EN ES

**MED***intim*  
personal healthcare

# Caya<sup>®</sup> *contoured diaphragm*

## **Instruction manual**

**Contraceptive diaphragm**  
Natural, safe, hormone-free

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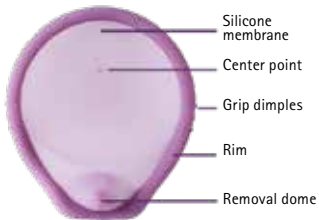
# Caya® *contoured diaphragm*

## Instruction manual

Your **Caya®** diaphragm is a safe and easy-to-use contraceptive method. Always use **Caya®** in combination with a contraceptive gel. The shape of the **Caya®** diaphragm is designed to fit the female anatomy. One size is sufficient and fits most women.


### Your Caya® diaphragm looks like this:

- **Silicone membrane** is placed over the cervix and prevents entry of the sperm cells into the uterus.
- **Center point:** palpable center point for better positioning of the diaphragm.
- **Grip dimples:** to facilitate vaginal insertion.
- **Flexible rim:** to ensure correct position.
- **Removal dome** on the front of the diaphragm to facilitate an removal from the vagina.

### What is the position of the Caya® diaphragm in the vagina?

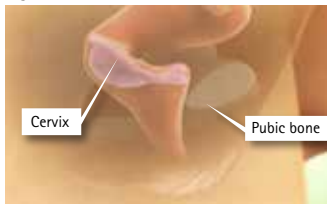When inserted correctly, the diaphragm covers the cervix and is positioned in the upper part of the vagina, between the pubic bone and the vaginal walls.

During and after sexual intercourse, the **Caya®** diaphragm prevents sperm cells from entering through the cervix into the uterus. This turn prevents unwanted pregnancies.

**NOTE:** Please always use the diaphragm together with a contraceptive gel.

Practice inserting the **Caya®** diaphragm on your own a few times before using it for the first time during intercourse.

### Before using Caya® the first time:

- Take your time
- Read the instruction manual carefully
- If possible, watch the animation on the internet at [www.caya.us.com](http://www.caya.us.com)

- Find the cervix – it feels like the tip of your nose
- Also find your pubic bone (in the vagina)

### How do I insert the Caya® diaphragm?

**NOTE:** Insert the **Caya®** diaphragm before intercourse.

1. Remove the **Caya®** diaphragm from the package with clean hands and wash it with water and soap before using it the first time.
2. Before inserting the **Caya®** diaphragm prior to intercourse, place 4 ml (about a teaspoon) of contraceptive gel into the silicone membrane.

Spreading the gel around the rim facilitates insertion into the vagina.

Before inserting it into the vagina, fold the **Caya**® diaphragm using the grip dimples.

**NOTE:** If you have inserted the diaphragm more than 2 hours before sex, you need to add more contraceptive gel.

3. Choose a position which is comfortable for you – kneeling down, lying down or standing with one leg elevated.

4. Hold the **Caya**® diaphragm with one hand. Place the thumb and index finger on the grip dimples along the rim. The arrow has to point towards your body.

5. Using your free hand, spread the labia. Push **Caya**® deeply along the posterior vaginal wall, towards your back, until the cervix rests inside the cup.

6. Use a finger to verify that the cervix is covered by the membrane of the **Caya**® diaphragm. Can you feel the cervix through the cup? If yes, then it is in the correct position.

If you cannot feel the cervix or the **Caya**®, remove the diaphragm and try inserting it again.

The edge of the removal dome should be located securely and safely behind the upper part of the pubic bone.

If you have verified both of the above and the diaphragm is in the correct position, safe contraception is ensured, you are now ready for intercourse.

7. After intercourse, the **Caya**® diaphragm must remain in the vagina for at least 6 hours. Do not leave the **Caya**® diaphragm in place longer than 24 consecutive hours.

**NOTE:** The **Caya**® diaphragm should never be left in the vagina for longer than 24 hours without interruption. Wearing it for longer, uninterrupted periods can promote bacterial growth in the vagina. These bacteria could lead to Toxic Shock Syndrome.

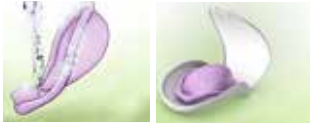
In the event of repeated intercourse, or if the **Caya**® diaphragm was inserted longer than 2 hours before intercourse, use a vaginal applicator to insert additional gel deep into the vagina without removing the diaphragm.

8. To remove the diaphragm, insert one finger into the vagina and pull out the **Caya**<sup>®</sup> diaphragm using the removal dome.

9. Cleaning and care of the **Caya**<sup>®</sup> diaphragm: After removing the diaphragm, clean it with water and soap and allow it to air-dry naturally. Store the **Caya**<sup>®</sup> diaphragm in the original case. This protects the diaphragm from external damage.

The **Caya**<sup>®</sup> diaphragm is reusable. After the first use you can use it for two years.

10. Please check your **Caya**<sup>®</sup> diaphragm for holes and tears before each use. Make sure that you do not damage it with your fingernails.

## About Caya<sup>®</sup>:

**Use:** The **Caya**<sup>®</sup> contoured diaphragm is indicated for the prevention of pregnancy in women who elect to use diaphragms as a method of contraception. It is recommended for use with a contraceptive gel.

The shape and safety of the **Caya**<sup>®</sup> diaphragm were developed in an intensive research process lasting several years in the USA and in worldwide studies.

- The **Caya**<sup>®</sup> diaphragm is a birth control method a woman puts in her vagina before she has sex.
- **Caya**<sup>®</sup> covers the cervix (the opening to the uterus). The **Caya**<sup>®</sup> diaphragm used with a contraceptive gel helps block sperm and helps keep the woman from getting pregnant.
- One size fits most women.
- Wait to remove **Caya**<sup>®</sup> at least six hours after intercourse.
- Never leave in the vagina for longer than 24 hours without interruption.

**NOTE:** Always keep the **Caya**<sup>®</sup> diaphragm in the original case and out of the reach of children.

## Information

Toll free phone  
**1-800-850-1657**  
[www.caya.us.com](http://www.caya.us.com)

## More Information:

- About 14 out of 100 women might get pregnant in one year, if they use **Caya**<sup>®</sup> the right way every time they have sex.
- About 18 out of 100 might get pregnant in a year, if they do not use **Caya**<sup>®</sup> the right way every time they had sex.
- If you are not sure that **Caya**<sup>®</sup> is a good method for you, talk with your doctor.
- You should replace **Caya**<sup>®</sup> every two years.

## Warnings – Do not use Caya® if you:

- Are within the first 6 weeks after delivery/birth.
- Previously used a traditional diaphragm of size 60 mm or smaller, or size 85 mm and larger.
- In case of a severely prolapsed pelvic floor/cervix.

**NOTE:** Never use the Caya® diaphragm with lubricant gels containing silicone.  
Intended for personal use only – not to share with others.

## Caya® does not prevent STDs or HIV/AIDS:

- You should use a condom along with Caya® to help protect yourself from these diseases.

## Talk to your healthcare provider, if:

- You have difficulty getting the Caya® diaphragm to fit in your body comfortably.
- You use treatments for vaginal infections.
- You frequently suffer from urinary tract infections (rare).
- You have had lower abdominal/vaginal surgery (very rare).
- You have ever had a Toxic Shock Syndrome (very rare).

## Stop using Caya® and talk to your healthcare provider, if:

- You or your partner get burning, itching, a rash, or other irritation of the vagina or penis.
- You or your partner have difficult or painful urination.

Please consult your healthcare provider for any other medical questions.

**NOTE:** If you have forgotten to use the Caya® diaphragm and have had unprotected sexual intercourse, please consider Emergency Contraception to prevent pregnancy. Take Emergency Contraception within 72 hours of unprotected intercourse.

**Material:**  
medical grade silicone rubber, nylon

**Contents:**  
– One Caya® vaginal diaphragm  
– Instructions for use  
– Storage case

## FAQs

### How effective is the Caya® diaphragm?

No contraceptive method can guarantee 100 % safety from unwanted pregnancy. The Caya® diaphragm is just as safe as the classic diaphragm. The more often you use the Caya® diaphragm, the more confident you will become in using this contraceptive method and the more safety this method will offer you.

### How can I positively influence safety when using the Caya® diaphragm?

Regular use of the Caya® diaphragm and the use of a contraceptive gel are important measures for ensuring contraceptive effectiveness. In addition, the diaphragm should always be inserted before and not during intercourse.

**How do I know that Caya® is inserted correctly and fits?**  
Use your longest finger (usually the middle finger) to verify the correct position of the diaphragm: check whether the cup covers cervix and the removal dome is located securely behind your pubic bone. If this is the case you are well protected.

### For how long can I use the Caya® diaphragm?

After the first use, the Caya® diaphragm can be used for 2 years before replacement.

### Why does the Caya® diaphragm have to remain in the vagina for 6 hours after intercourse?

Leaving Caya® in place for 6 hours will help ensure that sperm are inactivated by the contraceptive gel that you used with the Caya® diaphragm. This will help prevent unwanted pregnancy.

**Why should Caya® not be worn for longer than 24 hours without interruption?**  
Wearing the Caya® diaphragm for longer than 24 hours without interruption can promote bacterial growth inside the vagina. These bacteria can lead to Toxic Shock Syndrome.

### What should I do if I forget to remove Caya® after 24 hours have passed?

Remove Caya® and contact your healthcare provider regarding the long insertion period to see if an examination is necessary.

### Can I have repeated consecutive intercourse with the Caya® diaphragm?

Yes, your Caya® will remain in position, but you will have to apply additional contraceptive gel with the aid of an applicator. Applicators for contraceptive gel can be purchased in the pharmacy.

Can the Caya® diaphragm also be used during menstruation? The **Caya®** diaphragm is intended to be used for contraception. It is not intended for collection of menstrual blood.

#### What do I have to observe in case of a vaginal infection?

Please visit your healthcare provider for treatment in case of a vaginal infection (e.g. fungus). Please use condoms during treatment and NOT a diaphragm. Upon completion of the treatment you should purchase and use a new **Caya®** diaphragm for your own safety.

#### Could my partner feel the Caya® diaphragm?

This is highly unlikely. However, in rare cases, your partner may feel the rim of the diaphragm.

#### Does the Caya® diaphragm influence my sexual desire?

No, the **Caya®** diaphragm has no influence on your libido as it contains no hormones.

#### What should I do if Caya® causes pain during intercourse?

First verify the correct position of **Caya®** using a finger. If the **Caya®** diaphragm continues to cause pain, please visit your doctor.

#### What should I do if Caya® accidentally comes out during intercourse?

Re-insert **Caya®** again after applying some more contraceptive gel. If the diaphragm comes out after your partner has ejaculated or if you do not feel safe, visit your healthcare provider to discuss whether you may need an Emergency Contraception.

#### Are there restrictions for the use of the Caya® diaphragm?

No, the only habit that requires changing is that one always uses the **Caya®** diaphragm when it is needed – during intercourse. The more often **Caya®** is used, the more confident one becomes in handling the diaphragm.

#### Can Caya® break or tear?

This is very rare. However, if you do notice that your **Caya®** is damaged in some way after intercourse, please visit your healthcare provider immediately (within 72 hours) to discuss the possibility of emergency contraception ("morning-after pill").

#### Can I additionally use lubricants with Caya®?

Yes, but only water-soluble lubricants. **Important:** Do not use any silicone oil lubricants. However, we cannot make any statements about the compatibility of the lubricants with the contraceptive gels since we have no data available on this. That is, any limitations of the efficacy of the contraceptive gel cannot be ruled out.

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