

birth control you provide

HORMONE-FREE • SIMPLE • EFFECTIVE



3 WAYS TO PROVIDE CAYA TO YOUR PATIENTS

1 DISPENSE to patients directly

While you can prescribe Caya for your patients via pharmacies, the quickest and most efficient way for them to obtain it is directly from you during their appointment.

BENEFITS FOR YOUR PRACTICE:

- No minimum & easy ordering
- Educational info is included
- Increases practice income

BENEFITS FOR PATIENT:

- Increases patient comfort
- Learn insertion & usage
- No prescription delays



2 MAIL ORDER PHARMACY

Issue a Caya Diaphragm prescription for fulfillment via mail-order pharmacy with delivery directly to your patient's home.



3 LOCAL PHARMACY

Issue a Caya Diaphragm prescription for fulfillment and pick up at your patient's local pharmacy.

HOW TO ORDER CAYA TO DISPENSE:

NEW CUSTOMERS:

1. Fill out an application
2. Provide state license info
3. Receive a customer login

EXISTING CUSTOMERS:

1. Place order via online portal
2. Call or email to place order
3. Fax order to customer service



800-850-1657

customerservice@hpsrx.com


HPSRx Enterprises is the exclusive distributor of Caya in the USA. Our mission is to help expand healthcare options for women. HPSRX is licensed to ship to all 50 states.

PROVIDER QUICK REFERENCE GUIDE FOR COACHING

See our Caya Provider Brochure or website for comprehensive product and patient information: Caya.US.com/provider-information

Caya[®]

Contoured
diaphragm

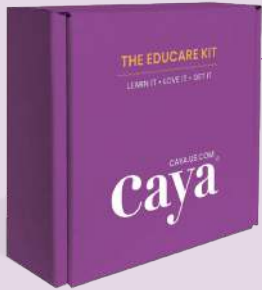

EDUCARE KIT: COACHING IN ONE BOX



Single-use diaphragm*



Patient quick-info brochure



Caya IFU brochure

Our **FREE Caya EduCare Kit** simplifies the process of teaching your patients about Caya. Each purple box has everything you need to:

- Educate about insertion and removal
- Show how to clean and care for Caya
- Provide take-home information

*Dispose of the single-use diaphragm (with the hole) after appointment. Do not let the patient take it home.



1. OPEN

Open the single-use diaphragm (with the hole) provided in the purple box.



2. HOLD

Hold the diaphragm by grip dimples, arrow pointed toward cervix.



3. APPLY

Squeeze Caya and apply a water-based contraceptive gel to the edges.



4. INSERT

With one hand open the labia and insert Caya with the other hand.



5. PUSH

Push anterior edge of removal dome up behind the pubic bone/pubic notch.



6. CHECK

Caya is correctly placed if cervix is completely covered by the diaphragm.




7. PATIENT

Have the patient check fit and practice inserting and removing it.



8. REMOVAL

Slip the finger into the removal dome, pull down, and remove Caya.

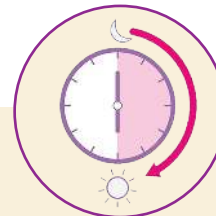

9. ADVISE

Caya must be left in place for 6 hours after having sex before removing.



10. DISPOSE

Throw away the diaphragm with hole. Give the EduCare box & brochures to patient.

ORDER CAYA EDUCARE KITS & PROVIDER MATERIALS:

For additional EduCare Kits, Provider Brochures, or patient education and display materials for your office, please contact us.